



Zekler 1500

en Instructions for Use
sv Bruksanvisning
da Brugsanvisning
no Bruksanvisning
fi Käytöohje
fr Notice d'utilisation
de Gebrauchsanweisung
no Bruksanvisning
pl Instrukcja obslugi

Zekler Safety
 52385 Ulrichshausen
 Sweden
 Phone +46 321 506 00
info@zekler.com
www.zekler.com

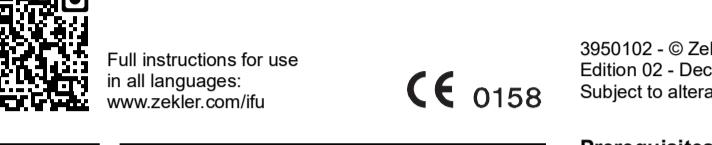

CE 0158

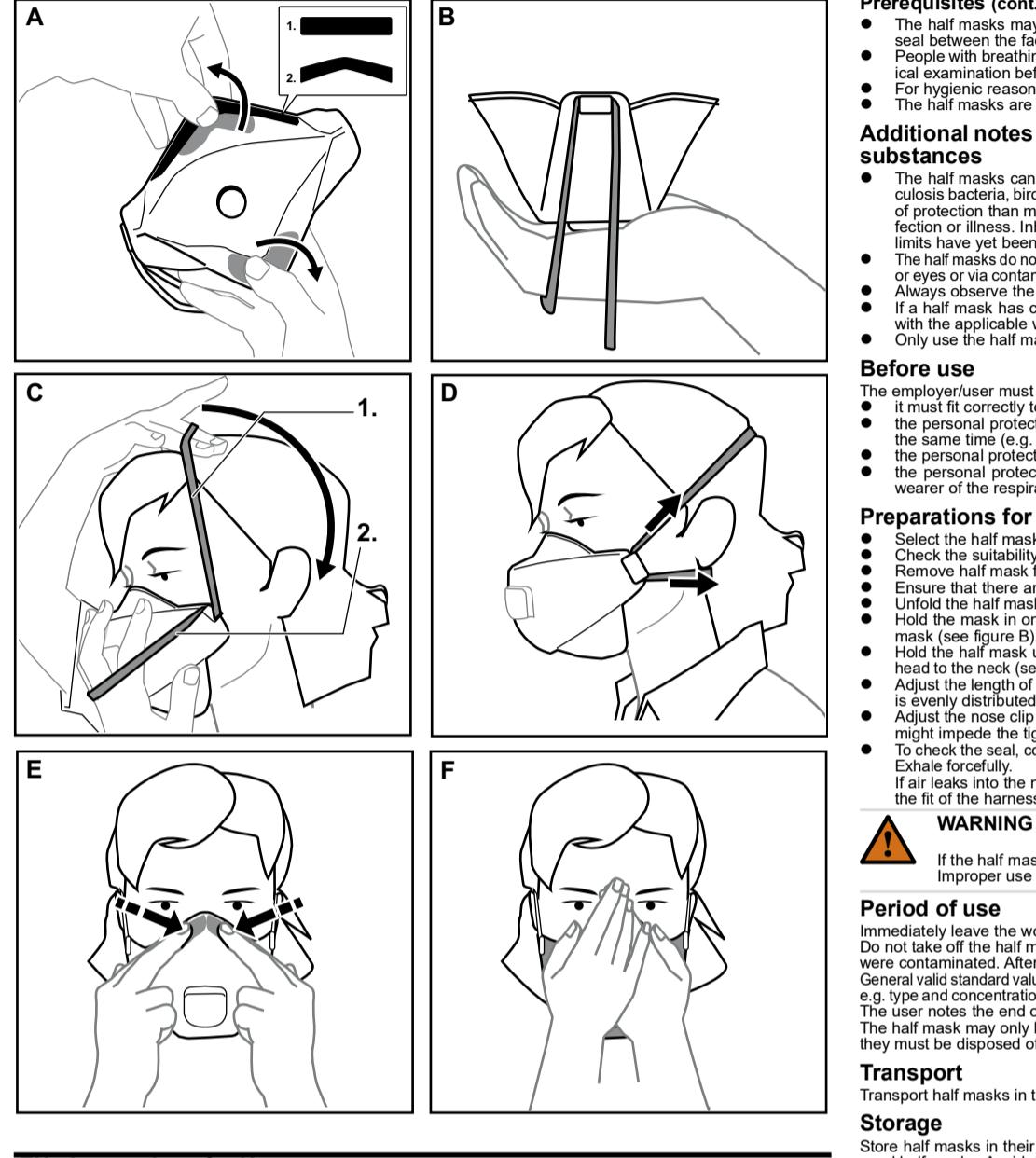**EN - Instructions for Use****Safety-related information**

Any use of particle filtering half masks requires full understanding and strict observation of these instructions for use. The particle filtering half masks are only to be used for the purpose specified here.

In case of queries please contact your safety officer, us or see EN 529:2005.

Description**Product overview**The Zekler® 1500 series¹⁾ includes the following particle filtering half masks (hereafter also called "half masks"):

• Zekler 1500, protection class FFP2

• Zekler 1503, protection class FFP3

Nägra av halvmaskerna har en utändningsventil som underlättar utändning.

Intended use**WARNING**

The particle filtering half masks are only to be used for the purpose specified here.

Misuse can cause illness or death!

Limitations in the intended use

The half masks must not be used against solid and/or liquid non-volatile particles resulting e.g. from grinding, polishing, brushing, sawing or mineral processing.

The oxygen content of the air must not drop below the following limit values:

17 Vol-% in the Netherlands, Belgium and the UK.

For other countries, observe the national regulations.

The half masks may not be used for a maximum one work shift (8 hours).

These half masks do not include complete warning and/or rescue equipment for use of personnel in the event of an emergency situation or biological harmful substances is possible.

Information on this topic is available from the hygiene officer or Zeiker Safety.

The maximum concentration of pollutants must be known. The maximum permitted concentration of the pollutants in the ambient air must not be exceeded. Observe the limit values and limitations of use for the various half masks.

Protection of the category**Multiplex²⁾ of the limit value³⁾****Restrictions on use⁴⁾**

FFP1 4 Not against carcinogenic and radioactive materials, air-borne biological substances of risk category 2 and 3 and enzymes

FFP2 10 To be used as protection from carcinogenic and radioactive materials, air-borne biological substances of risk category 3, and enzymes only after a risk assessment

FFP3 30⁴⁾

1) In accordance with EN 529:2005, amendments by national regulations are possible.

2) Maximum permitted concentration of harmful substances specified by country.

3) Any divergence from any diverging national regulations.

4) UK: Not applicable

• General note on the use of particle filtering half masks: asbestos, silica, sandblast particles, lead, arsenic, 4th metyldietylin (MDA), cadmium and cancerforming alkalis have särskilt hög skadlig effekt.

• Partikelfiltrerande halvmasker är partiellt kanterade och är inte tillräckligt för att skydda mot vissa partiklar.

• Detta gäller för de delar som är tillverkade i plast och som är försedda med filter.

• Detta gäller för de delar som är tillverkade i plast och som är försedda med filter.

• Kontakt Zeiker Safety vid eventuell användning i områden med explosionssfärer.

• Apparater som är godkända enligt EN 149:2001+A1:2009 och (EU) 2016/425.

Declaration of conformity: see www.zekler.com/doc

• Important! Strictly follow the instructions for use.

D Variant that passed the dolomite clogging test.

• Storage until ... NR Use only for a maximum of one work shift (8 hours)!

• Temperature range of storage conditions V Half mask with exhalation valve

• Maximum humidity of storage conditions S, M/L Size of the half mask

Use**Prerequisites****WARNING**

Half masks are not to be used if there is any confusion about the purpose of their use or the conditions in which they are to be used. Please observe the following notes during use. Misuse can cause illness or death!

• Users must be familiar with the use. Observe the national rules and regulations on the use of respiratory equipment.

• It must be ensured that the ambient atmosphere cannot change unfavorably.

• Protect the half mask from moisture and dirt and store it in the original packaging.

• The filter must not be damaged or destroyed by handling or storage.

• Do not use damaged or used half masks or half masks from damaged packaging.

• Do not modify half masks.

1) Zeiker® is a registered trademark of Zeiker Safety.

Notified body involved in type approval:
 Institut für Arbeitsschutz der Deutschen Gesetzlichen Unfallversicherung (IFA)
 Alte Heerstraße 111
 53757 Bonn
 Germany
 Reference number: CE 0121

involved in quality control:
 DEKRA Testing and Certification GmbH
 Hansestrasse 15
 70565 Stuttgart
 Germany
 Reference number: CE 0158

Pull instructions for use in all languages:
www.zekler.com/fu

CE 0158

3950102 - © Zekler Safety

Edition 02 - December 2023

Subject to alteration

